
















DESIGNS FOR VISION

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Nike Skylon Ace		Skylon Ace		DFV-TTL-Skylon Ace	<ol style="list-style-type: none"> 1.Remove T-bracket from T-mount. 2.Remove screws from T-bracket. 3.Insert T-bracket into the hole on the nose bridge of your frame. 4.From the back of the nose bridge, thread the screws through the frame and back into the T-bracket. 5.Tighten down screws until T-bracket is secure. 6.Slide T-mount down over T-bracket. Done!
Sky View		Sky View		DFV-Skyview	<ol style="list-style-type: none"> 1.Ensure the grey Linkage Arm is oriented on top. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. Done!
Nike Pounce		Nike Pounce /Revolve		DFV-TTL-	
Nike Revolve		Nike Pounce /Revolve		DFV-TTL-	

DESIGNS FOR VISION

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Nike Curfew		Modified PI - Round		DFV-Curfew	<ol style="list-style-type: none"> 1.Remove screw from mount. 2.Ensure the silver washer and one black washer are on the screw. 3.Insert screw (with a washer on it) into the gap in the back of the nose bridge on your frame. 4.Thread screw into mount. Mount should be on the front of your frame. 5.Tighten down screw. Done!
Buddy Holly /Yeoman		U-Bracket		DFV-TTL-Yeoman	<ol style="list-style-type: none"> 1.Back out the set screw until it is flush with the inside of the mount. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Designer 109		PI-Round		DFV-Curfew	<ol style="list-style-type: none"> 1.Remove screw from mount. 2.Ensure the silver washer and one black washer are on the screw. 3.Insert screw (with a washer on it) into the gap in the back of the nose bridge on your frame. 4.Thread screw into mount. Mount should be on the front of your frame. 5.Tighten down screw. Done!
Titanium 062		DFV 062		DFV-TITANIUM-062	<ol style="list-style-type: none"> 1.Detach mounting plate from mount. 2.Place the mounting plate on the inside of the frame on the nose bridge. 3.Place mount on the outside of the frame. Thread the screw from the mounting plate into the mount. 4.Tighten screw. Done!
Designer 047		DFV-NIKE-Curfew		DFV-TTL-BlackMount	<ol style="list-style-type: none"> 1.Remove the screw from the mount. After removing the screw, ensure the silver washer and 1 black washer are on the screw. 2.Insert the screw (with the washer on it) into the gap on the back of the nose bridge on your frame.. 3.Thread the screw through the other washer and

DESIGNS FOR VISION

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
					into the mount. 4.Tighten the screw down. Done!
Sport Frame		Nike sport		DFV-DVI-Sport	1.Remove T-bracket from T-mount. 2.Remove screws from T-bracket. 3.Insert T-bracket into the hole on the nose bridge of your frame. 4.From the back of the nose bridge, thread the screws 5.through the frame and back into the T-bracket. 6.Tighten down screws until T-bracket is secure. 7.Slide T-mount down over T-bracket. Done!
Small Sport Frame		Nike sport		DFV-DVI-Sport	1.Remove T-bracket from T-mount. 2.Remove screws from T-bracket. 3.Insert T-bracket into the hole on the nose bridge of your frame. 4.From the back of the nose bridge, thread the screws 5.through the frame and back into the T-bracket. 6.Tighten down screws until T-bracket is secure. 7.Slide T-mount down over T-bracket. Done!
Nike Run		Nike Run		DFV-NIKE-Run	1.Ensure the grey Linkage Arm is oriented on the top. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. Done!
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
DFV- T connector				Design for vision T-connector to the light	DFV- T connector for attaching light to loupe