

EXAMVISION

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Exam Vision		EV-TTL-2 holes		EXAMV-2REARHOLES	<ol style="list-style-type: none"> 1.Remove screws from the mount. 2.With the grey Linkage Arm down, line up the holes on the mount with the holes on the frame. 3.Insert screws upwards (from below mount) through the holes on the frame and into the mount. Tighten down. Done!
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Exam Vision Light					