





MERIDENT OPTERGO

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Bolle		Merident basic		Merident loupe attachment	<ol style="list-style-type: none"> 1.Remove the standard parts behind the camera 2.Take the Merident connection plates (loupe mount) and place them like shown in the photo below 3. push them together and turn the part shown on right counter clockwise, so that the small hole comes visible through the bigger hole 4.Place the Merident connection set on the back of the camera and use the original screws to tighten it on its place. Done! (The camera can now be “clipped” into place)
VinKep / TLL		2-holes model		Merident Vinkep 2 holes	
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
MO Light			