




















ORASCOPTIC

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Rudy Project - Rydon		TTL 2 holes		ORA-TTL-2 Holes	<ol style="list-style-type: none"> 1. Remove cover from mounting plate on your loupes. 2. With the grey linkage arm oriented on top, place the mount over the 2 screw holes on your frame. 3. Tighten screws down. 4. Done!
Victory Titanium		TTL 2 holes		ORA-TTL-2 Holes	<ol style="list-style-type: none"> 1. Remove cover from mounting plate on your loupes. 2. With the grey linkage arm oriented on top, place the mount over the 2 screw holes on your frame. 3. Tighten screws down. 4. Done!
Standard01 Titanium		U-bracket		ORA-TTL-2 Holes	<ol style="list-style-type: none"> 1. Remove cover from mounting plate on your loupes. 2. With the grey linkage arm oriented on top, place the mount over the 2 screw holes on your frame. 3. Tighten screws down. 4. Done!
Square Crossbar		Medium Flip-Up		ORA-FP-GEM M	<ol style="list-style-type: none"> 1. Ensure the grey Linkage Arm is oriented on top. 2. Slide mount onto crossbar. 3. Done!
GEM		Small Flip-Up		ORA-FP-GEM S	<ol style="list-style-type: none"> 1. Ensure the grey Linkage Arm is oriented on top. 2. Slide mount onto crossbar. 3. Done!

ORASCOPTIC

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
XV1		XV1 U-bracket (battery version)		ORA-TTL-XV1	
Standard02 Titanium		XV1 U-bracket		ORA-TTL-XV1	<ol style="list-style-type: none"> 1. Back out the set screw until it is flush with the inside of the mount. 2. Slide the nose bridge (frame) into the opening of the U-bracket mount. 3. Tighten down set screw once mount is in place. 4. Done!
Legend		XV1 U-bracket		ORA-TTL-XV1	<ol style="list-style-type: none"> 1. Back out the set screw until it is flush with the inside of the mount. 2. Slide the nose bridge (frame) into the opening of the U-bracket mount. 3. Tighten down set screw once mount is in place. 4. Done!
Rave		XV1 U-bracket		ORA-TTL-XV1	<ol style="list-style-type: none"> 1. Back out the set screw until it is flush with the inside of the mount. 2. Slide the nose bridge (frame) into the opening of the U-bracket mount. 3. Tighten down set screw once mount is in place. 4. Done!
ErgoEdge Sport		ORA-ErgoEdge		ORA-ErgoEdge	<ol style="list-style-type: none"> 1. Remove cover from mounting plate on your loupes. 2. With the grey linkage arm oriented on top, place the mount over the 2 screw holes on your frame. 3. Tighten screws down. 4. Done!

ORASCOPTIC

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Rudy Project -Revolution		TTL 2 holes		ORA-TTL-2 Holes	<ol style="list-style-type: none"> 1. Remove cover from mounting plate on your loupes. 2. With the grey linkage arm oriented on top, place the mount over the 2 screw holes on your frame. 3. Tighten screws down. 4. Done!
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Orasoptic Light v1		Orasoptic Light			3mm gap
Orasoptic Light v2		Orasoptic Light			4mm gap