

PERIOPTIX

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Advista		Advista		PER-TTL-Advista	<ol style="list-style-type: none"> 1. Hook the larger arm of the mount over the top of the frame. 2. Push down on the grey linkage arm to snap the mount into place. Done!
Ultralight Titanium		U-Bracket		PER-TTL-Titanium	<ol style="list-style-type: none"> 1. Back out the set screw until it is flush with the inside of the mount. 2. Slide the nose bridge (frame) into the opening of the U-bracket mount. 3. Tighten down set screw once mount is in place. Done!
Panoramic	 Hogies Advista T-Sight Golf	Small Flip-Up		PER-Flip-up	<ol style="list-style-type: none"> 1. Ensure the grey Linkage Arm is oriented on top. 2. Slide mount onto crossbar. Done!
Legacy (square cross bar)	 Hogies Rimz Mirage Ultralight Titanium	8mm Flip-Up		PER-Flip-up	<ol style="list-style-type: none"> 1. Ensure the grey Linkage Arm is oriented on top. 2. Slide mount onto crossbar. Done!
Panoramic	 Hogies	Small Flip-Up		PER-Flip-up	<ol style="list-style-type: none"> 1. Ensure the grey Linkage Arm is oriented on top. 2. Slide mount onto crossbar. Done!