

SURGITEL

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Oakley Half Jacket		TTL Half Jacket		Surgitel TTL- Oakley Half Jacket large U-bracket mount.	1.Back out the set screw until it is flush with the inside of the mount. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Surgitel Dovetail		Oakley M		SUR-TTL-Oakley M	1.Back out the set screw until it is flush with the inside of the mount. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Oakley frame 2.0		Flak Jacket 2.0		SUR-TTL-Flak 2.0	1.Back out the set screw until it is flush with the inside of the mount.2. Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Oakley M-frame		SUR-TTL-Oakley M		Surgitel TTL - All frames with SG plate	1.Back out the set screw until it is flush with the inside of the mount. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Oakley Radar		SG Plate		SUR- TTL-SG Plate	1.Ensure grey Linkage Arm is up. 2.Slide mount onto mounting plate, ensuring the Linkage Arm is up. 3.Push down into place. Done!


SURGITEL

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Dove Tail		TTL-Dovetail		SUR-TTL-Dovetail	<ol style="list-style-type: none"> 1.Ensure the round side of the dovetail mount is facing downward. 2.Insert the Dovetail mount into the frame round side down. Done! Make sure it's in there all the way.
Standard Titanium		PI-Round			
Ergo Titanium		PI-Round		DFV-Curfew	<ol style="list-style-type: none"> 1.Remove screw from mount. 2.Ensure the silver washer and one black washer are on the screw. 3.Insert screw (with a washer on it) into the gap in the back of the nose bridge on your frame. 4.Thread screw into mount. Mount should be on the front of your frame. 5.Tighten down screw. Done!
Aero Titanium		TTL-Aero		SUR-TTL-Aero	<ol style="list-style-type: none"> 1.Back out the set screw until it is flush with the inside of the mount. 2.Slide the nose bridge (frame) into the opening of the U-bracket mount. 3.Tighten down set screw once mount is in place. Done!
Other Oakley		Universal Clip		Universal Clip	(included in the standard sale package)

SURGITEL

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Micro Ocular		Micro Flip-Up		Surgitel - Flip up - Cross bar	<p>1.Ensure the grey Linkage Arm is oriented on top. 2.Slide mount onto crossbar. Done!</p> <p>(Note: Flip up mounts are available in different size, such as micro / small / medium/large. Please contact to inquire if you are not sure.)</p>
Ultralight Ocular		Medium Flip-Up		Surgitel - Flip up - Cross bar	<p>1.Ensure the grey Linkage Arm is oriented on top. 2.Slide mount onto crossbar. Done!</p> <p>(Note: Flip up mounts are available in different size, such as micro / small / medium/large. Please contact to inquire if you are not sure.)</p>
Prism Ocular		Medium Flip-Up		Surgitel - Flip up - Cross bar	<p>1.Ensure the grey Linkage Arm is oriented on top. 2.Slide mount onto crossbar. Done!</p> <p>(Note: Flip up mounts are available in different size, such as micro / small / medium/large. Please contact to inquire if you are not sure.)</p>
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Surgitel light connector		Surgitel Light		Surgitel Light	<p>Installing the camera to Surgitel (futudent.com)</p>