
















UNIVET

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Sport Techne Galilean		TTL Techne		UNI-TTL-TEC	1. Remove screw on the top of the nose bridge with the included torque screwdriver. 2. Insert mount into the screw hole on the nose bridge. 3. Tighten the screw on the mount. Make sure the slot on mount is facing forward.
Techne RX Prismatic		TTL Techne RX		UNI-TTL-TEC	1. Remove screw on the top of the nose bridge with the included torque screwdriver. 2. Insert mount into the screw hole on the nose bridge. 3. Tighten the screw on the mount. Make sure the slot on mount is facing forward.
Univet Air-X		Univet Magnetic		UNI-Magnetic	
Urban Life Univet ITA		TTL 062		TTL-062	1. unscrew the mounting plate from the mount. 2. Place the mounting plate on the side of the frame with the screw pointing forward (same direction as the oculars). 3. Place the mount onto the front of the mounting plate (Linkage Arm up).
Standard sport		Snap-on		Snap-on	1. Hook the top arm of the mount over the top of the nose bridge. 2. Push the mount down onto the frame until it snaps in place. Done!

UNIVET

Loupe Attachment Library of Novocam Medical Innovations Oy

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Univet01 Frame		TTL U-Bracket		TTL-U-Bracket	1. Back out set screw until it flush with mount 2. Slide mount onto frame. Done!
Univet ONE		Large U-Bracket		Large U-Bracket	
Univet Iris		Short U-bracket		Short U-bracket	
LIGHT	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
Univet Magnetic		Univet Magnetic	 	LA lens light adaptor	Slide it to camera light joint.