

FRAME	PHOTO	MOUNT PART	MOUNT PHOTO	NOVOCAM SKU	INSTRUCTION
EyeMag Pro		CZ Pro		Zeiss-EyeMag-Pro	<ol style="list-style-type: none"> <li>1. Insert the 2 screws on the mount into the 2 holes on the hinge of your loupes.</li> <li>2. Tighten the screws down.</li> <li>3. Slide the camera onto the mount. Done!</li> </ol> <p>Note: if you are using the Zeiss EyeMag Pro S (headgear), you will need to remove and re-insert the screw in the opposite direction and turn the grey Linkage Arm to face the opposite way. After changing their direction, then follow the above steps.</p>
EyeMag Smart		CZ Smart		Zeiss-EyeMag-Smart	<ol style="list-style-type: none"> <li>1. Slide the mount onto the center bar of your loupes.</li> <li>2. Slide the camera to the attaching item.</li> </ol>
EyeMag-Pro+light		Zeiss-EyeMag-Pro (with light installed)		Zeiss-EyeMag-Pro+light	